

1.5T ADNI GE 11.0 M4 Software, TwinSpeed Gradient and Single-Channel Birdcage Head Coil

Set up the patient according to the ADNI instruction manual.

Use the electronic ADNI protocol that has been loaded on your scanner.

NOTE: The "adni" pulse sequence used for series 2 and 3 is a works in progress pulse sequence that ADNI sites obtain from GE Healthcare.

SERIES	1. 3 plane loc.	scan plane mode SAT	<i>IMAGING PARAMETERS</i>		<i>ACQUISITION TIMING</i>	
	coil HEAD		3-plane	matrix/nex	256 / 128 / 1	
	etl		(Whole Body gradient)	fov (cm)	26	
	scan time :16			slice/space	5 / 5	
			autoshim	On		
comments	The preferred coil is the 8-channel brain coil. This protocol is for sites with single channel birdcage coils only.					

SERIES	2. Sag MPRAGE	scan plane mode pulse seq image opts. psd name	<i>IMAGING PARAMETERS</i>		<i>ACQUISITION TIMING</i>	
	coil HEAD		Sag	matrix/nex	192 / 192 / 1	
	#echos		3D (ZOOM gradient)	phase fov		
	te		SPGR	locs/pause		
Prep time	<i>SCAN TIMING</i>	EDR, IrPrep, Fast	freq. direct.	S/I		
flip angle	1	adni	fc direct			
etl	min full	<i>ADDITIONAL PARAMETERS</i>	phase corr			
bw1/bw2	1000		autoshim	On		
scan time	8		Filter Choices = none	<i>SCANNING RANGE</i>		
	15.63		turbo mode = 1	fov	24	
	9:36	swap fat chem shift = 1	slice/space	1.2mm 184 locs/slab		
		mprage mode = 1				
		TD or mprageTR=3000				
comments	Cover skin to skin. Remind the patient to hold still for this scan.					

SERIES	3.MPRAGE-repeat	scan plane mode pulse seq image opts. psd name	<i>IMAGING PARAMETERS</i>		<i>ACQUISITION TIMING</i>	
	coil HEAD		Sag	matrix/nex	192 / 192 / 1	
	#echos		3D (ZOOM gradient)	phase fov		
	te		SPGR	locs/pause		
Prep time	<i>SCAN TIMING</i>	EDR, IrPrep, Fast	freq. direct.	S/I		
flip angle	1	adni	fc direct			
etl	min full	<i>ADDITIONAL PARAMETERS</i>	phase corr			
bw1/bw2	1000		autoshim	Off		
scan time	8		Filter Choices = none	<i>SCANNING RANGE</i>		
	15.63		turbo mode = 1	fov	24	
	9:36	swap fat chem shift = 1	slice/space	1.2mm 184 locs/slab		
		mprage mode = 1				
		TD or mprageTR=3000				
comments	Prescribe same image locations as series 2, unless adjustment is needed (e.g. to correct for wrap). Remind the patient to hold still for this scan.					

(Continued on next page)

1.5T ADNI GE 11.0 M4 Software, TwinSpeed Gradient and Single-Channel Birdcage Head Coil

SERIES

coil	4. Ax PD/T2 FSE HEAD	scan plane mode	Ax 2D (ZOOM gradient)	matrix/nex	256 / 256 / 1
	<i>SCAN TIMING</i>	pulse seq	FSE-XL	phase fov	0.9
#echoes	2	image opts.	EDR, Fast	acqs/pause	0
te	min full / TE2=100	psd name		freq. direct.	A/P
TR	3000		<i>ADDITIONAL PARAMETERS</i>	fc direct	
flip angle		User CVs	Filter Choices=none	Autoshim	Off
etl	16		blurring cancellation=0	phase corr	
bw1/bw2	20.83				<i>SCANNING RANGE</i>
scan time	6:25			fov	24
				slice/space	48 loc, 3mm interleaved
comments	Prescribe 48 slices to cover head.				

Series 4 is the final patient series.

Follow the ADNI instructions to complete the phantom scans.