

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ADNI2_Siemens_3T_TrioTim_VB17b\ADNI2_Siemens_3T_TrioTim_VB17\HUMAN\localizer

TA: 0:10 PAT: Off Voxel size: 1.9x1.5x8.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	single

Phase resolution	75 %
Phase partial Fourier	Off
Interpolation	Off

PAT mode	None
Matrix Coil Mode	Auto (CP)

Image Filter	Off
Distortion Corr.	Off
Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off
Raw filter	On
Intensity	Weak
Slope	25
Elliptical filter	Off

Routine

Slice group 1	
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 2	
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Slice group 3	
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Rotation	90.00 deg
Phase oversampling	0 %
FoV read	280 mm
FoV phase	100.0 %
Slice thickness	8.0 mm
TR	20.0 ms
TE	5.00 ms
Averages	1
Concatenations	3
Filter	Raw filter, Prescan Normalize
Coil elements	HEA;HEP

Geometry

Multi-slice mode	Sequential
Series	Ascending

Saturation mode	Standard
Special sat.	None

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

Body	Off
HEP	On
HEA	On

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
R >> L	350 mm
A >> P	263 mm
F >> H	350 mm

Contrast

TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	40 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Physio

Tagging	None
Dark blood	Off

Resp. control	Off

Resolution

Base resolution	192
-----------------	-----

Inline

Subtract	Off
Liver registration	Off

SIEMENS MAGNETOM TrioTim syngo MR B17

Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off

MapIt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Off
Bandwidth	180 Hz/Px
Flow comp.	No
Allowed delay	0 s

Segments	1
RF pulse type	Normal
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ADNI2_Siemens_3T_TrioTim_VB17b\ADNI2_Siemens_3T_TrioTim_VB17\HUMAN\MPRAGE

TA: 9:14 PAT: Off Voxel size: 1.0x1.0x1.2 mm Rel. SNR: 1.00 SIEMENS: tfl

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	176
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	1.20 mm
TR	2300 ms
TE	2.98 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast

Magn. preparation	Non-sel. IR
T1	900 ms
Flip angle	9 deg
Fat suppr.	None
Water suppr.	None
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off
PAT mode	None
Matrix Coil Mode	Auto (CP)
Image Filter	Off
Distortion Corr.	Off
Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off

Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Single shot
Series	Interleaved
Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System

Body	Off
HEP	On
HEA	On
Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Sagittal
Rotation	0.00 deg
F >> H	256 mm
A >> P	240 mm
R >> L	212 mm

Physio

Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Off
Bandwidth	240 Hz/Px
Flow comp.	No
Echo spacing	7.1 ms

SIEMENS MAGNETOM TrioTim syngo MR B17

RF pulse type
Gradient mode
Excitation
RF spoiling

Fast
Normal
Non-sel.
On

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ADNI2_Siemens_3T_TrioTim_VB17b\ADNI2_Siemens_3T_TrioTim_VB17\HUMAN\MPRAGE

TA: 5:12 PAT: 2 Voxel size: 1.1x1.1x1.2 mm Rel. SNR: 1.00 SIEMENS: tfl

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	176
FoV read	270 mm
FoV phase	93.8 %
Slice thickness	1.20 mm
TR	2300 ms
TE	2.95 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast

Magn. preparation	Non-sel. IR
T1	900 ms
Flip angle	9 deg
Fat suppr.	None
Water suppr.	None

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Matrix Coil Mode	Triple
Reference scan mode	Integrated

Image Filter	Off
Distortion Corr.	Off

Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Single shot
Series	Interleaved

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System

Body	Off
HEP	On
HEA	On

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Sagittal
Rotation	0.00 deg
F >> H	270 mm
A >> P	254 mm
R >> L	212 mm

Physio

Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Off

SIEMENS MAGNETOM TrioTim syngo MR B17

Bandwidth	240 Hz/Px
Flow comp.	No
Echo spacing	7 ms

RF pulse type	Fast
Gradient mode	Normal
Excitation	Non-sel.
RF spoiling	On

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ADNI2_Siemens_3T_TrioTim_VB17b\ADNI2_Siemens_3T_TrioTim_VB17\HUMAN\Axial T2-FLAIR

TA: 4:05 PAT: 2 Voxel size: 0.9x0.9x5.0 mm Rel. SNR: 1.00 SIEMENS: tse

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	35
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Rotation	90.00 deg
Phase oversampling	0 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	5.0 mm
TR	9000 ms
TE	90.0 ms
Averages	1
Concatenations	3
Filter	Distortion Corr.(2D), Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast

TD	0.0 ms
MTC	Off
Magn. preparation	Slice-sel. IR
TI	2500 ms
Freeze suppressed tissue	Off
Flip angle	150 deg
Fat suppr.	None
Water suppr.	None
Restore magn.	Off

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
Phase resolution	100 %
Phase partial Fourier	Off
Trajectory	Cartesian
Interpolation	Off

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	47
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated

Image Filter	Off

Distortion Corr. Mode	On
Unfiltered images	2D
Unfiltered images	Off
Prescan Normalize	Off
Normalize	On
B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Multi-slice mode	Interleaved
Series	Interleaved

Special sat.	None

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

Body	Off
HEP	On
HEA	On

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
A >> P	220 mm
R >> L	220 mm
F >> H	175 mm

Physio

Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

SIEMENS MAGNETOM TrioTim syngo MR B17

Sequence

Introduction	On
Dimension	2D
Compensate T2 decay	Off
Reduce Motion Sens.	Off
Contrasts	1
Bandwidth	222 Hz/Px
Flow comp.	No
Allowed delay	30 s
Echo spacing	8.19 ms

Define	Turbo factor
Turbo factor	19
Echo trains per slice	8
RF pulse type	Fast
Gradient mode	Fast

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ADNI2_Siemens_3T_TrioTim_VB17\ADNI2_Siemens_3T_TrioTim_VB17\HUMAN\AXIAL_T2_STAR

TA: 4:11 PAT: Off Voxel size: 1.0x0.8x4.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	44
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Rotation	90.00 deg
Phase oversampling	0 %
FoV read	200 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	650 ms
TE	20.00 ms
Averages	1
Concatenations	2
Filter	Distortion Corr.(2D), Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast

TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
Phase resolution	75 %
Phase partial Fourier	Off
Interpolation	Off

PAT mode	None
Matrix Coil Mode	Auto (CP)

Image Filter	Off
Distortion Corr. Mode	On 2D
Unfiltered images	Off
Unfiltered images	Off
Prescan Normalize	On
Normalize	Off

B1 filter	Off
Raw filter	Off
Elliptical filter Mode	On Inplane

Geometry

Multi-slice mode	Interleaved
Series	Interleaved

Saturation mode	Standard
Special sat.	None

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

Body	Off
NE2	Off
HEP	On
HEA	On
SP4	Off
SP2	Off
SP8	Off
SP6	Off
SP3	Off
SP1	Off
SP7	Off
SP5	Off

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	P >> A
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
R >> L	350 mm
A >> P	263 mm
F >> H	350 mm

Physio

Tagging	None
Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off

SIEMENS MAGNETOM TrioTim syngo MR B17

Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off

MapIt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Off
Bandwidth	200 Hz/Px
Flow comp.	Slice/Read
Allowed delay	20 s

Segments	1
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ADNI2_Siemens_3T_TrioTim_VB17b\ADNI2_Siemens_3T_TrioTim_VB17\HUMAN\ASL PERFUSION

TA: 6:02 PAT: Off Voxel size: 4.0x4.0x4.0 mm Rel. SNR: 1.00 SIEMENS: ep2d_pasl

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	24
Dist. factor	25 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	P >> A
Rotation	180.00 deg
Phase oversampling	0 %
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	3400.0 ms
TE	12 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast

Inversion time 2	1900.0 ms
Inversion time 1	700 ms
Saturation stop time	1800 ms
Flip angle	90 deg
Fat suppr.	Fat sat.
Fat sat. mode	Strong

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	105
Delay in TR	0 ms
Multiple series	Off

Perfusion mode	PICORE Q2T
Inversion time 1	700 ms
Saturation stop time	1800 ms
Inversion time 2	1900.0 ms
Flow limit	100.0 cm/s

Resolution

Base resolution	64
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

PAT mode	None
Matrix Coil Mode	Auto (CP)

Prescan Normalize	On
Raw filter	Off
Elliptical filter	Off

Hamming Off

Geometry

Multi-slice mode	Interleaved
Series	Ascending

Special sat.	Parallel F
Gap	25.4 mm
Thickness	100 mm

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System

Body	Off
HEP	On
HEA	On

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	180.00 deg
R >> L	256 mm
A >> P	256 mm
F >> H	119 mm

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Starting ignore meas	1
Ignore after transition	0
Model transition states	Off
Temp. highpass filter	On
Threshold	4.00
Paradigm size	4
Meas[1]	Baseline
Meas[2]	Active
Meas[3]	Baseline
Meas[4]	Active
Motion correction	On
Interpolation	3D-K-space
Spatial filter	On
Filter setting	2.0

Sequence

Introduction	On
Bandwidth	2368 Hz/Px
Free echo spacing	Off
Echo spacing	0.49 ms

SIEMENS MAGNETOM TrioTim syngo MR B17

EPI factor
RF pulse type
Gradient mode

64
Normal
Fast

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ADNI2_Siemens_3T_TrioTim_VB17b\ADNI2_Siemens_3T_TrioTim_VB17\HUMAN\Field_mapping

TA: 2:07 Voxel size: 1.7x1.7x3.0 mm Rel. SNR: 1.00 SIEMENS: gre_field_mapping

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	45
Dist. factor	25 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Rotation	90.00 deg
Phase oversampling	0 %
FoV read	212 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
TR	488 ms
TE 1	4.92 ms
TE 2	7.38 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast

MTC	Off
Flip angle	60 deg
Fat suppr.	None
Averaging mode	Long term
Reconstruction	Phase
Measurements	1
Multiple series	Off

Resolution

Base resolution	128
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off
Matrix Coil Mode	Auto (CP)
Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Interleaved
Series	Interleaved
Special sat.	None

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System

Body	Off
HEP	On
HEA	On
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
A >> P	212 mm
R >> L	212 mm
F >> H	168 mm

Composing

Sequence

Introduction	On
Dimension	2D
Asymmetric echo	Off
Contrasts	2
Bandwidth	260 Hz/Px
Flow comp.	Yes
RF pulse type	Normal
Gradient mode	Normal
RF spoiling	On

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ADNI2_Siemens_3T_TrioTim_VB17b\ADNI2_Siemens_3T_TrioTim_VB17\PHANTOM\localizer

TA: 0:10 PAT: Off Voxel size: 1.9x1.5x8.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	single

Phase resolution	75 %
Phase partial Fourier	Off
Interpolation	Off

PAT mode	None
Matrix Coil Mode	Auto (CP)

Image Filter	Off
Distortion Corr.	Off
Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off
Raw filter	On
Intensity	Weak
Slope	25
Elliptical filter	Off

Routine

Slice group 1	
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 2	
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Slice group 3	
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Rotation	90.00 deg
Phase oversampling	0 %
FoV read	280 mm
FoV phase	100.0 %
Slice thickness	8.0 mm
TR	20.0 ms
TE	5.00 ms
Averages	1
Concatenations	3
Filter	Raw filter, Prescan Normalize
Coil elements	HEA;HEP

Geometry

Multi-slice mode	Sequential
Series	Ascending

Saturation mode	Standard
Special sat.	None

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

Body	Off
HEP	On
HEA	On

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
R >> L	350 mm
A >> P	263 mm
F >> H	350 mm

Contrast

TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	40 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Physio

Tagging	None
Dark blood	Off

Resp. control	Off

Resolution

Base resolution	192
-----------------	-----

Inline

Subtract	Off
Liver registration	Off

SIEMENS MAGNETOM TrioTim syngo MR B17

Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off

MapIt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Off
Bandwidth	180 Hz/Px
Flow comp.	No
Allowed delay	0 s

Segments	1
RF pulse type	Normal
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ADNI2_Siemens_3T_TrioTim_VB17b\ADNI2_Siemens_3T_TrioTim_VB17\PHANTOMMPRAGE
 TA: 5:12 PAT: 2 Voxel size: 1.1x1.1x1.3 mm Rel. SNR: 1.00 SIEMENS: tfl

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	176
FoV read	270 mm
FoV phase	93.8 %
Slice thickness	1.30 mm
TR	2300 ms
TE	2.95 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast

Magn. preparation	Non-sel. IR
T1	900 ms
Flip angle	9 deg
Fat suppr.	None
Water suppr.	None

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Matrix Coil Mode	Triple
Reference scan mode	Integrated

Image Filter	Off
Distortion Corr.	Off

Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Single shot
Series	Interleaved

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System

Body	Off
HEP	On
HEA	On

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Sagittal
Rotation	0.00 deg
F >> H	270 mm
A >> P	254 mm
R >> L	229 mm

Physio

Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Off

SIEMENS MAGNETOM TrioTim syngo MR B17

Bandwidth	240 Hz/Px
Flow comp.	No
Echo spacing	7 ms

RF pulse type	Fast
Gradient mode	Normal
Excitation	Non-sel.
RF spoiling	On

SIEMENS MAGNETOM TrioTim syngo MR B17

Table of contents

\\USER				
	ADNI2_Siemens_3T_TrioTim_VB17b			
		ADNI2_Siemens_3T_TrioTim_VB17		
			HUMAN	
				localizer
				MPRAGE
				MPRAGE GRAPPA2
				Axial T2-FLAIR
				AXIAL_T2_STAR
				ASL PERFUSION
				Field_mapping
			PHANTOM	
				localizer
				MPRAGE Phantom GRAPPA2